


MetraBag

CONTROLLED TISSUE MORCELLATION
IN AN ENCLOSED SYSTEM

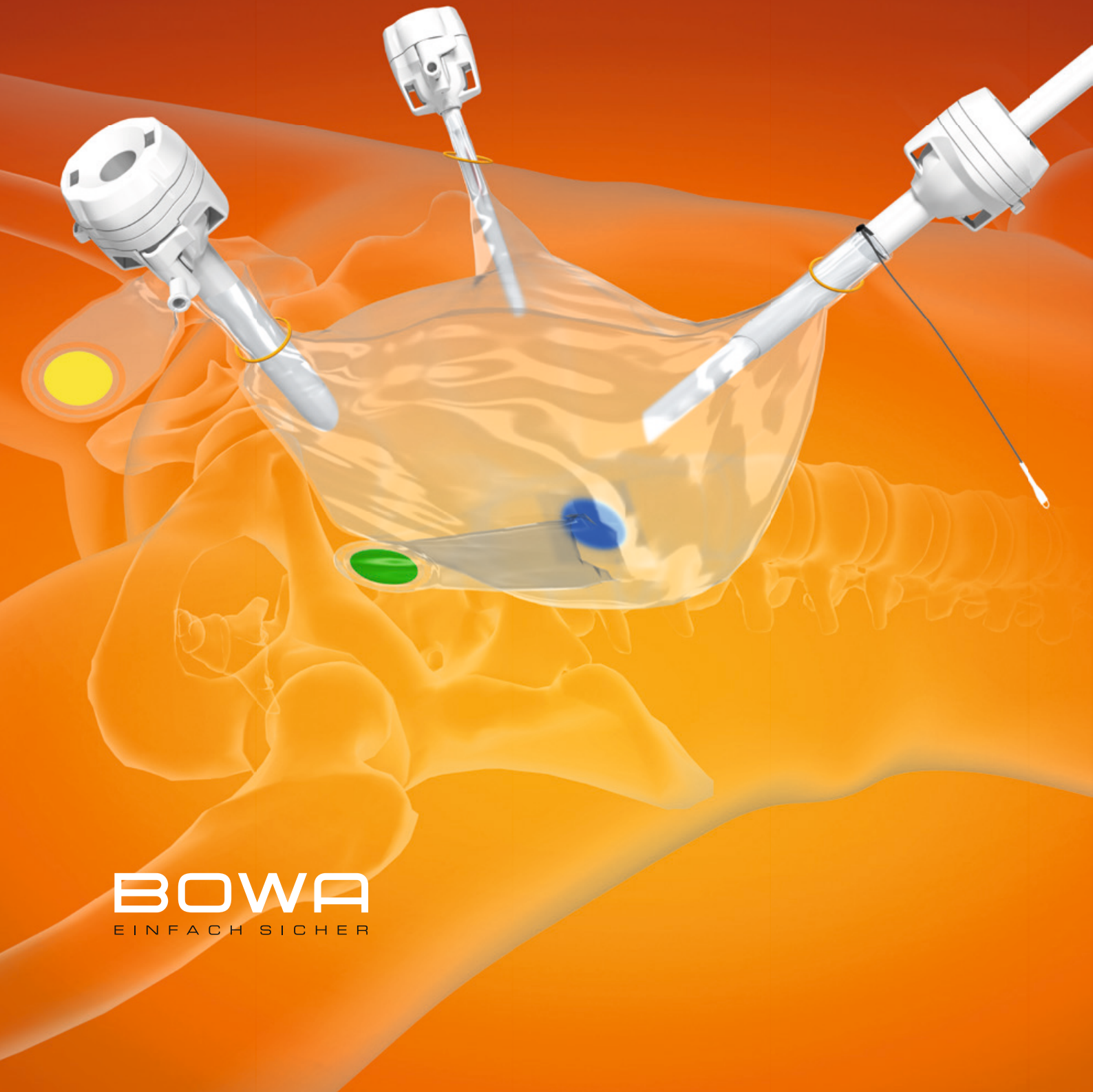

BOWA
EINFACH SICHER



MetraBag

Minimally invasive approaches have become established as the gold standard in the majority of surgical fields over the last 20 years and more. Suitable isolation bags and morcellation bags must be available so that patients can continue to benefit extensively from this minimally invasive technique.

Hysterectomies and myomectomies are among the most common operations performed in the

field of gynaecology. To ensure safe morcellation with minimal cell dispersion and containment of the excised material, BOWA is now offering a system that allows morcellation in an enclosed space.

The MetraBag morcellation bag from BOWA offers a solution for controlled tissue morcellation in an enclosed system.

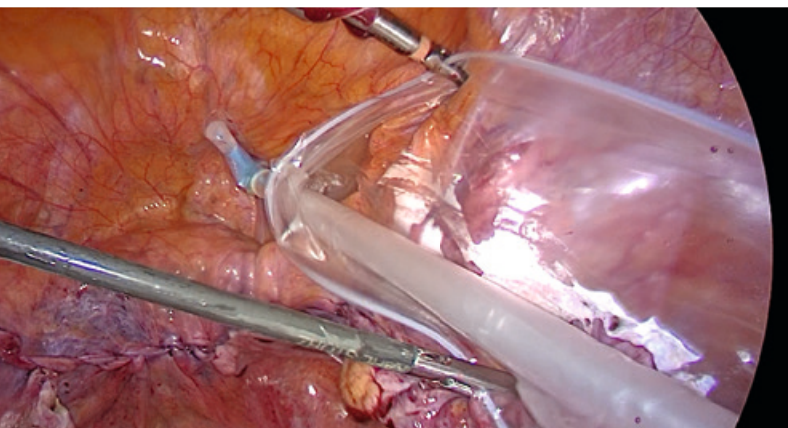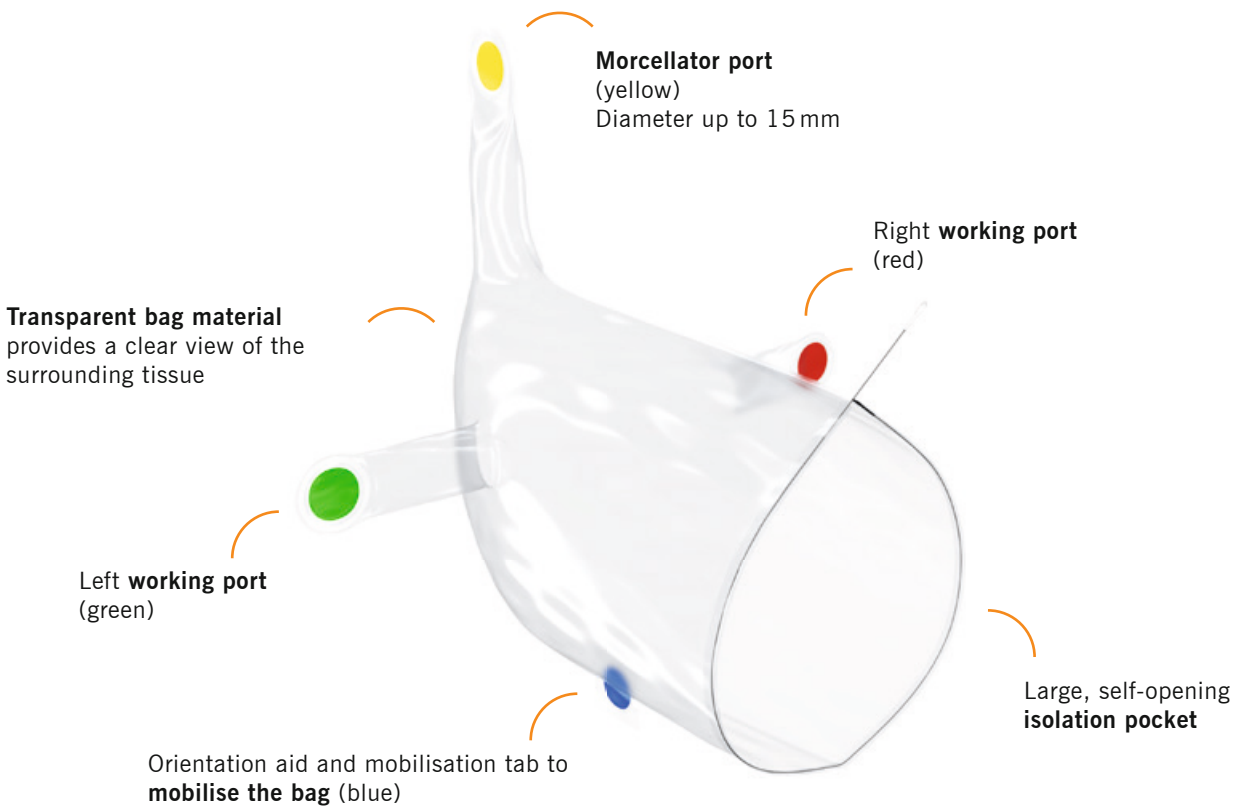


Intraoperative unrolling of the MetraBag




Positioning of the surgical site to be used

Application

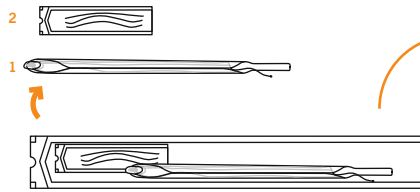


MetraBag
comprising MetraBag
with insertion aid and
three drawstrings


- (A)** Isolation opening with drawstring
- (B)** Central port (yellow)
- (C)** Left port (green)
- (D)** Right port (red)

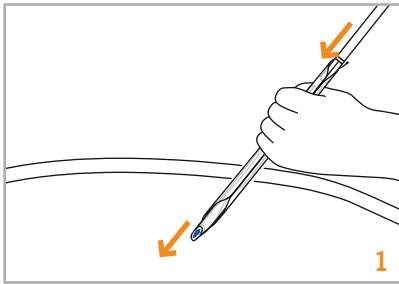


1 Insertion of the MetraBag using atraumatic grasping forceps

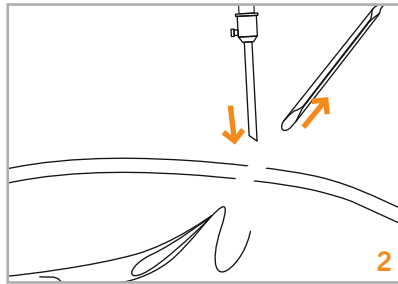


2 Removal of insertion aid and insertion of trocar

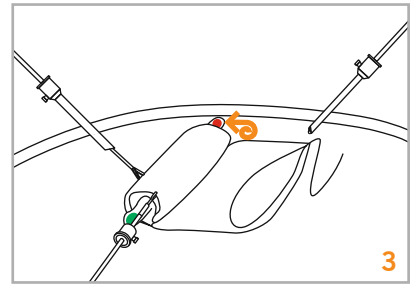


3 Unrolling the MetraBag

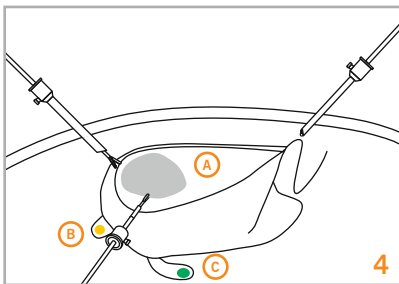


4 Containment of excised material in the MetraBag

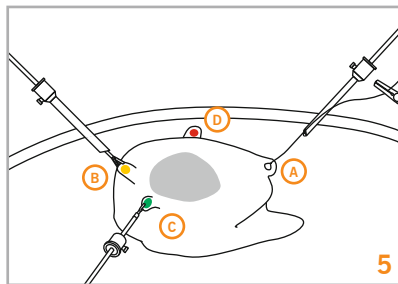


5 Sealing of the MetraBag with drawstring

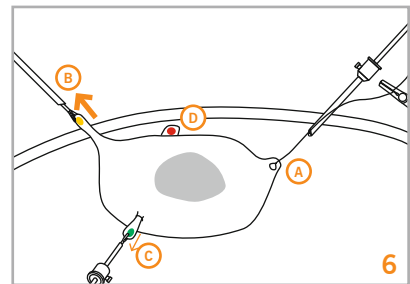


6 Positioning of the working port

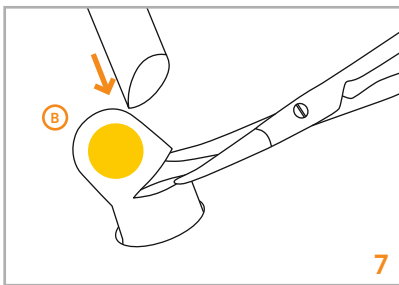


7 Opening of the extracorporeal working ports

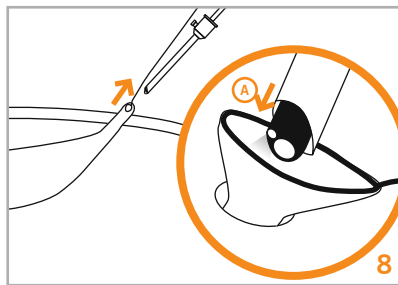


8 Extracorporeal positioning of the camera port and insertion of the laparoscope

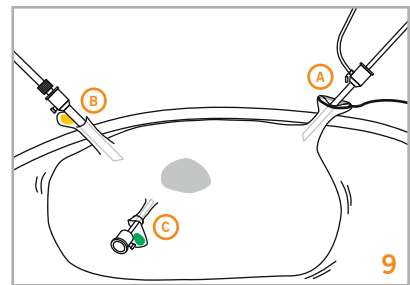


9 Insufflation of the MetraBag via trocar

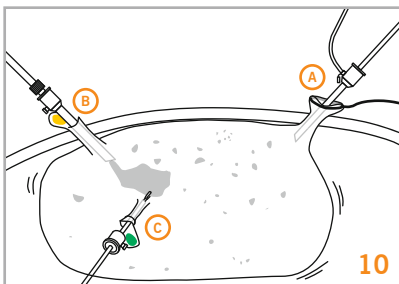

10 Morcellation of the excised material

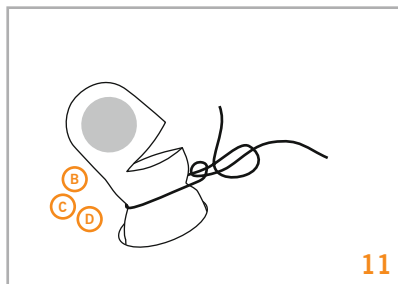

11 Sealing of used ports with suture materials

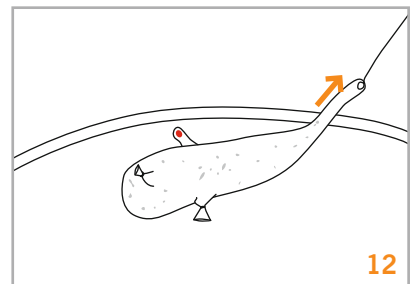


12 Extraction of the MetraBag from abdominal cavity

For more detailed information on the use of the MetraBag, please consult the MetraBag Instructions for Use REF 905-710.

Product features

- CO₂ insufflation of MetraBag creates ample working space.
- The transparent material provides a clear view of the surrounding tissue in the abdominal cavity.
- Colour-coding of the ports facilitates optimal positioning of the bag.
- Fluids and residual tissue are contained in the enclosed system.
- Sealable openings allow safe extraction of the bag.
- MetraBag can be used with different commercially available morcellators.

Ordering data

MetraBag	REF
	905-710
<p>Comprising:</p> <ul style="list-style-type: none">- Morcellation bag- Insertion aid- 3 drawstrings <p>Sterile 1 pack of ten</p>	

BOWA
EINFACH SICHER

BOWA-electronic GmbH & Co. KG
Heinrich-Hertz-Strasse 4 – 10
72810 Gomaringen | Germany

Phone +49 (0) 7072-6002-0
Fax +49 (0) 7072-6002-33
info@bowa.de | bowa-medical.com

