# BOWA

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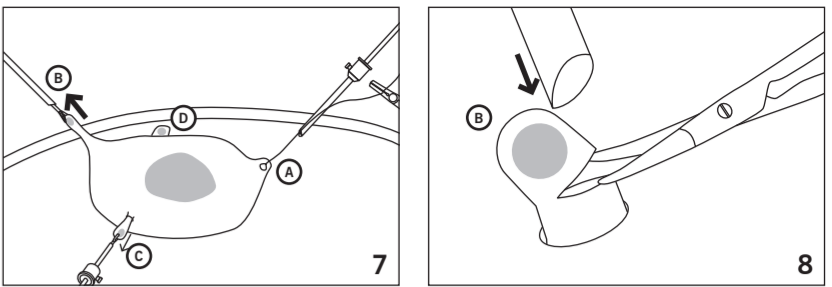
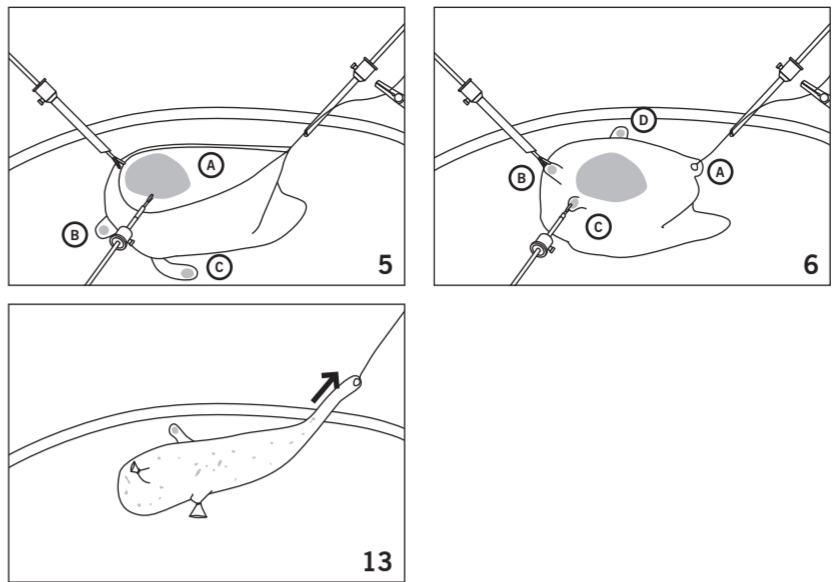
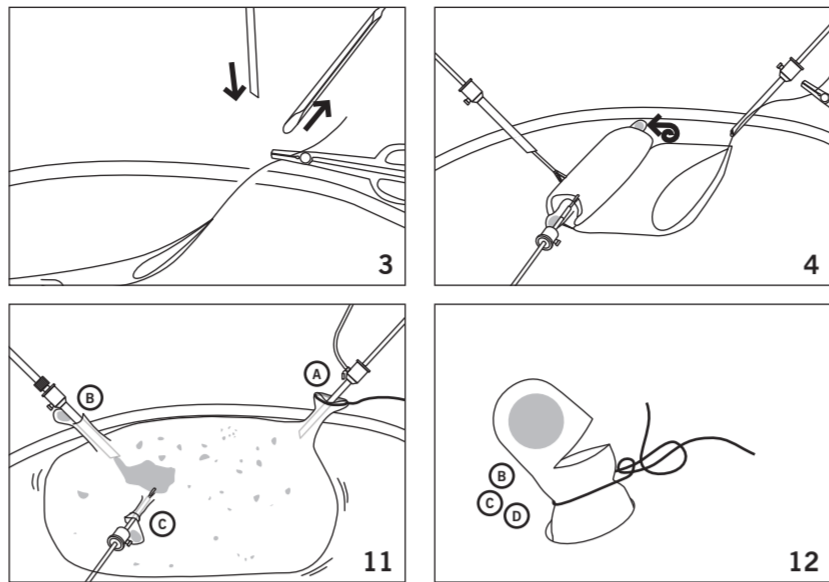
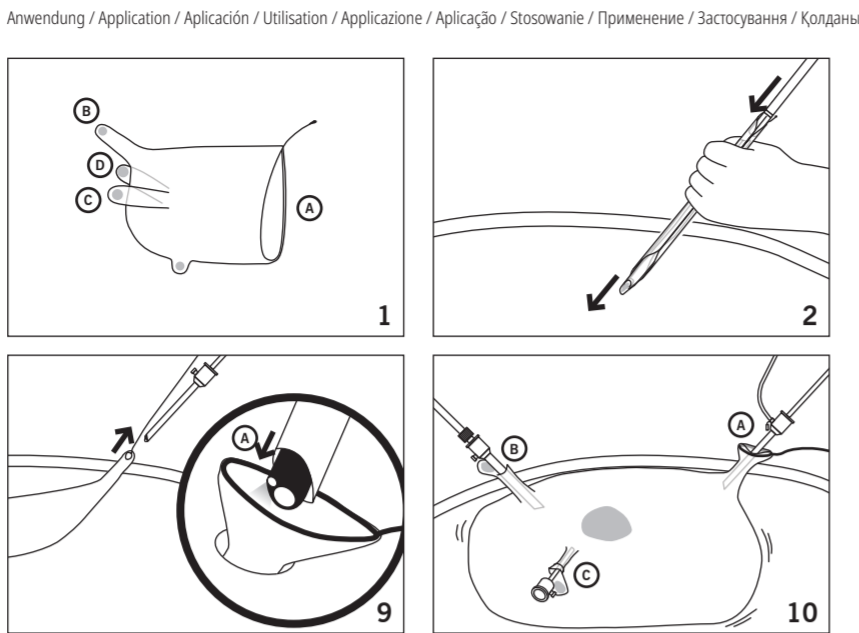
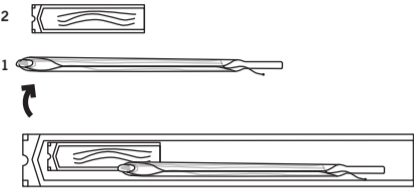
BOWA-electronic GmbH & Co. KG  
Heinrich-Hertz-Strasse 4 – 10  
72810 Gomaringen | Germany

Phone +49 (0) 7072-6002-0  
Fax +49 (0) 7072-6002-33  
info@bowa.de | bowa-medical.com

Lieferumfang / Delivery / Volumen de suministro / Livraison / Contenuto della confezione / Conteúdo fornecido / Zakres dostawy / Комплект поставки / Обсяг поставки / Сату және жеткізу көлемі:

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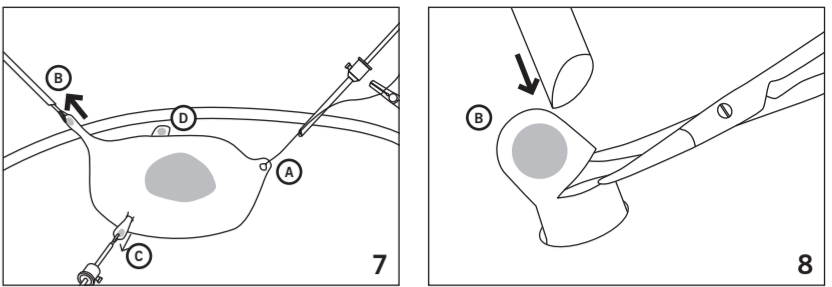
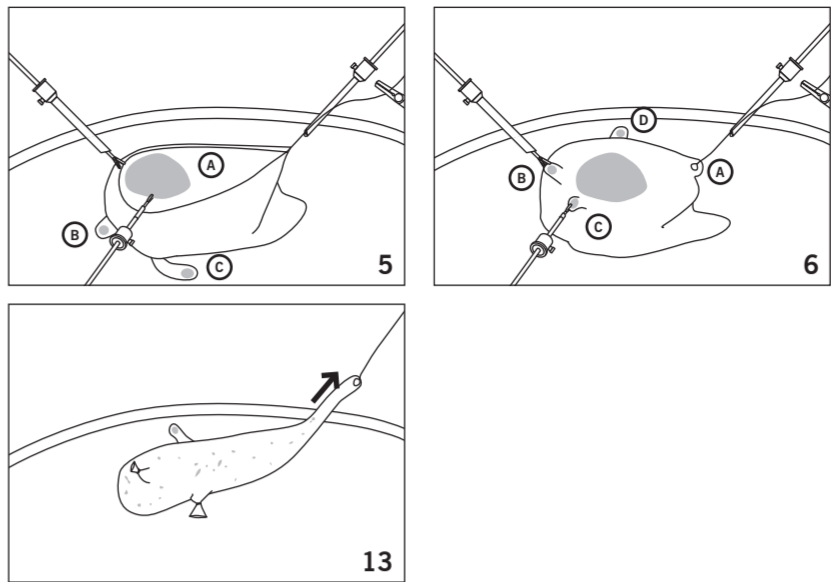
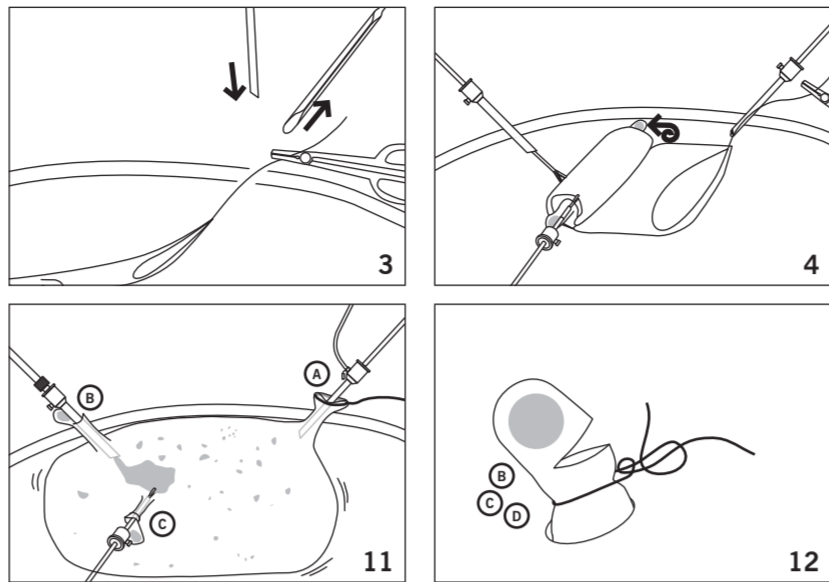
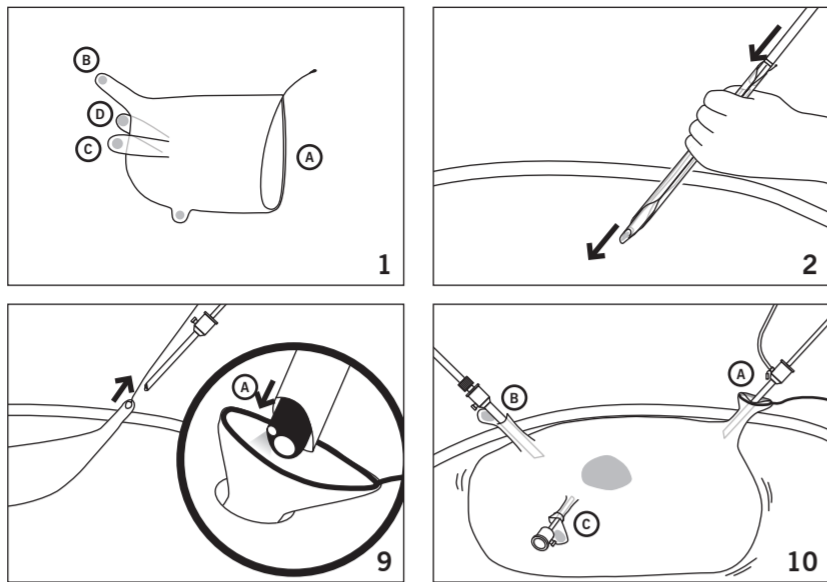
### EN INSTRUCTION FOR USE MetraBag – Morcellation bag REF 905-710

Symbol	Meaning	Symbol	Meaning
	Consult instructions for use		Do not use if packaging is damaged
	Caution, see documentation supplied with the product		Do not resterilize
	Catalogue number		Keep away from sunlight
	Batch		Keep dry
	Quantity		Temperature limitation
	Manufacturer		CE mark and identification number of notified body
	Sterilized using ethylene oxide		Conforms to Russian standards
	Expiry date		Conforms to Ukraine standards
	Single use only		

#### Delivery:

- MetraBag (1) with introducer
- Thread material (2) for closing the ports

#### IMPORTANT!

Please read all information carefully.

#### 1 INTENDED USE

The MetraBag is intended for laparoscopic morcellation of tissue in a closed system. The MetraBag is used to reduce intraoperative tissue spreading.

#### 2 CONTRAINDICATIONS

The MetraBag is not intended for use in non-laparoscopic procedures.

Do not use the medical product if, in the opinion of an experienced physician or according to current professional literature, such use would cause harm to the patient, for example due to the general condition of the patient, or if other contraindications are present.

#### 3 SAFETY INSTRUCTIONS

- Product and packaging must be checked for integrity and damage before use.
- Do not use damaged products.
- Product is only sterile when the packaging is undamaged.
- Do not use this product if the expiry date has been exceeded.
- Use an aseptic technique to remove the product from the package.
- Additional sterilizations are not permitted. This product changes its material properties if it is re-sterilized.
- BOWA expressly warns against modifying the product. Any modification leads to the exclusion of liability by BOWA.

#### 3.2 APPLICATION RELATED

- This product must only be used by trained medical personnel who have been instructed in its use on the basis of the instructions for use.
  - Only work under clear view.
  - Do not use any RF-applicators for morcellation.
  - Hot Instrument tips must not touch the morcellation bag.
  - Only atraumatic instruments should be used.
  - Use the enclosed thread material to close the ports of the morcellation bag.
  - After removing of the morcellation bag checks that all ports still closed.
- All serious incidents associated with the product should be reported to the manufacturer and the competent authority of the Member State in which the user is located.

#### 4 APPLICATION

- The MetraBag has four access points, through which instruments with up to a 15 mm diameter can be used. (A) Retrieval opening with shutter strap (B) Middle port (yellow) (C) Left port (green) (D) Right port (red)
- The MetraBag, located in the introducer, is inserted into an incision and placed in the surgical field using a suitable instrument (such as grasping forceps).
- The shutter strap of the MetraBag remains extracorporeal and can be locked with a clamp.
- Using a laparoscopic view and atraumatic laparoscopic instruments, the MetraBag is deployed and positioned.
- The mobilized tissue is recovered and placed in the MetraBag.
- The MetraBag is closed with the shutter strap.
- The required ports are inserted under laparoscopic view.
- The extracorporeal morcellation access port (B) is cut so that the morcellation trocar can be introduced. This applies equally to the optional selectable access ports (C) and (D).
- The closed retrieval opening of the MetraBag is pulled out by the shutter strap extracorporeally and opened so that the introduction of the camera trocar under view is possible.
- The MetraBag is now positioned and is insufflated by one of the used trocars, thereby lining the abdominal cavity.
- Morcellate the tissue in the MetraBag.
- All working access ports must be closed before removal of the MetraBag by means of enclosed thread material and use of at least 2 knots.
- The completely closed MetraBag is removed by means of shutter straps (A).

#### 5 OPERATING, STORAGE AND TRANSPORT CONDITIONS

ENVIRONMENTAL CONDITIONS	APPLICATION	TRANSPORT AND STORAGE
Temperature	+10 °C to +40 °C	-20 °C to +50 °C
Relative humidity	0 to 75 %	0 to 75 %
Atmospheric pressure	500 to 1060 hPa	500 to 1060 hPa

- Before opening a protective packaging, make sure that the packaging is clean.
- If the protective outer packaging is opened, it must be immediately closed & re-stored in a clean & dust free environment.
- If the product is removed from the protective packaging it must not be re-placed. A proper reclosure must be ensured.

#### 6 DISPOSAL

The medical product, packaging material and accessories must be disposed of in accordance with any applicable country-specific regulations and laws.

**MEDICAL PRODUCT GUIDELINE**  
THIS PRODUCT COMPLIES WITH GUIDELINE MDR 2017 / 745 FOR MEDICAL PRODUCTS.

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