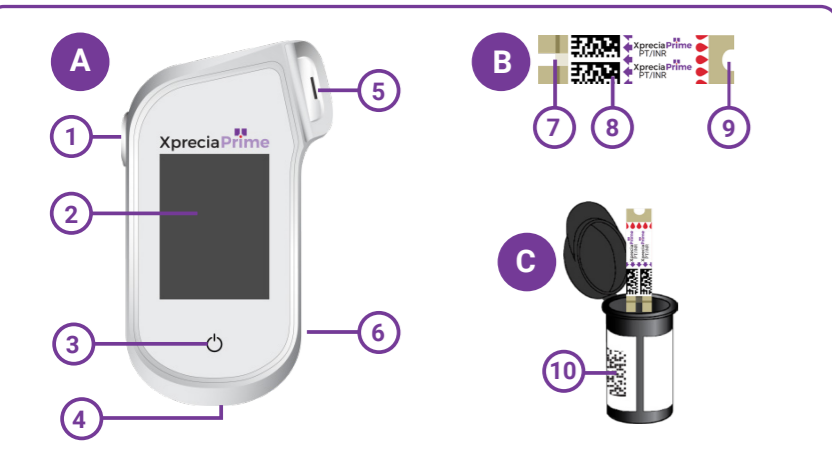
Patient Test

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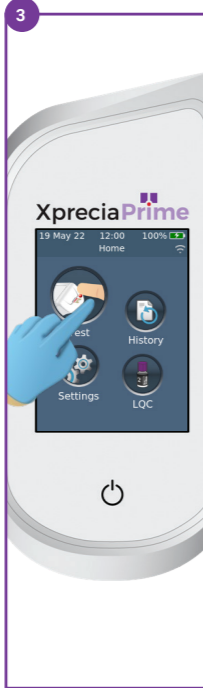
(EN) 1. WASH HANDS Wash your hands and the patient's hands with soap, then dry. 2. TURN ANALYZER ON Hold finger on power button to turn analyzer on.

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(EN) 3. TAP TEST BUTTON Tap Test button to begin. Wear a new pair of gloves for each test. 4. PATIENT ID Enter Patient ID by barcode (tap the on-screen barcode button) or on-screen keyboard.

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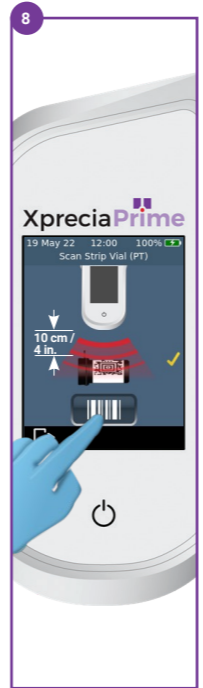
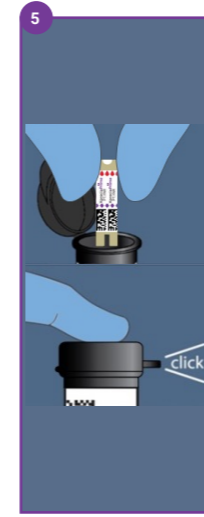
(EN) 5. GET STRIP Take out strip, then close vial firmly so the lid clicks. 6. SCAN STRIP Tap the on-screen barcode button to scan. Hold the barcode 10 cm (4 inches) from bottom of analyzer.

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(EN) 7. INSERT STRIP Insert strip when prompted. Insert with arrows pointing towards analyzer. 8. SCAN VIAL NOTE: This step only happens if you are using a new strip lot.

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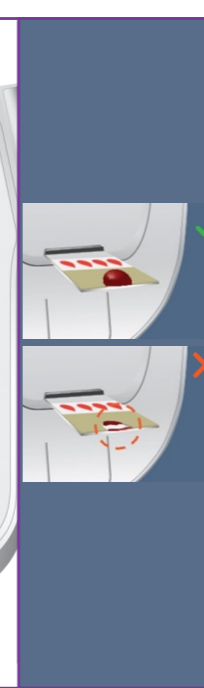
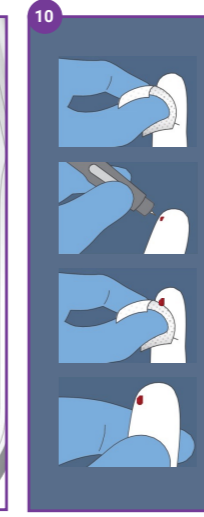
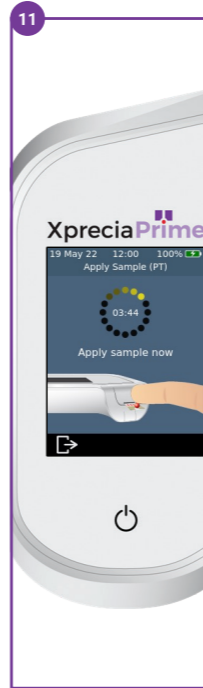
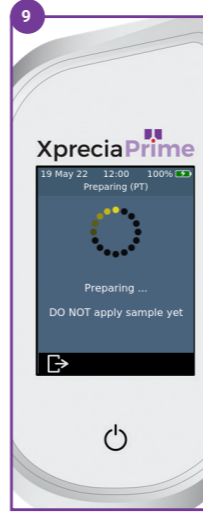
(EN) 9. WAIT Wait for analyzer to prepare. 10. OBTAIN SAMPLE Clean the finger with alcohol swab. Lance the finger (avoid fingers that have evidence of previous lancing). IMPORTANT: Wipe first drop with sterile gauze, use second drop.

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(EN) 11. APPLY SAMPLE Apply within 30 secs of fingerstick. Sample must fill whole of white semicircle.

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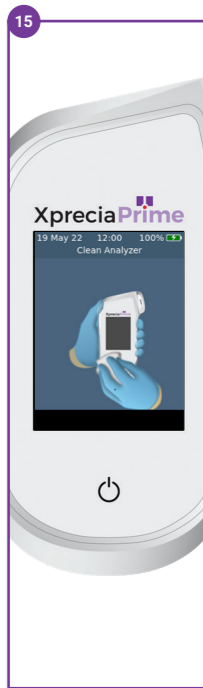
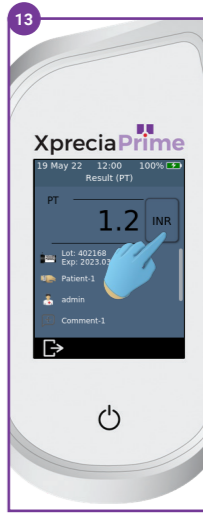
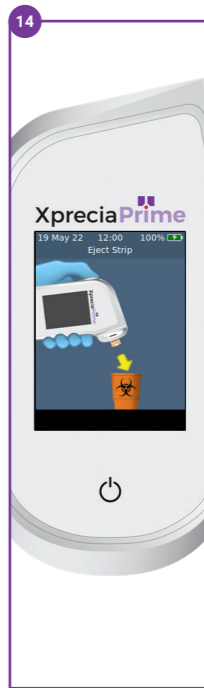
(EN) 12. WAIT Wait for analysis to complete. 13. RESULT Tap the units button to change units. Tap the exit button (bottom left) to continue.

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(EN) 14. EJECT STRIP Eject strip as shown above. All materials must go in a biohazard bin. 15. CLEAN ANALYZER Turn over this sheet for cleaning instructions.

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(EN) WARNINGS & GUIDANCE:

- Always wear gloves when cleaning – the analyzer might be contaminated
- Clean after each and every test to ensure the analyzer is always safe to use
- Use only CaviWipes™ (to avoid damaging your analyzer)
- Make sure the CaviWipe™ is damp (not dripping): always squeeze out excess liquid into a biohazard bin
- Never let fluid gather around or enter the strip port – always keep the strip port pointing downwards during cleaning
- Before cleaning, disconnect the USB cable (if connected)

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(EN) **A. TURN ANALYZER OFF** Hold finger on power button and select power down to turn analyzer off. **B. QUICK WIPE OF ANALYZER** Use a fresh CaviWipe™ and then dispose of the wipe. **C. REMOVE STRIP PORT COVER.**

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(EN) **D. THOROUGHLY CLEAN ANALYZER** Use a fresh CaviWipe™ - squeeze or twist to remove excess liquid (damp not dripping). Do four vertical and four horizontal strokes on each surface (two up and down; two left and right)

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(EN) **E. CLEAN STRIP PORT COVER** Pay special attention to strip port and channels inside cover. **F. WAIT 2 MINUTES IMPORTANT:** The disinfecting properties of CaviWipes™ need this time to take effect

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(EN) **G. DRY ANALYZER AND STRIP PORT.** Dry the whole analyzer and strip port cover using a lint free tissue/cloth. Use the same motions as Step D paying special attention to areas where fluid can gather. **H. REASSEMBLE STRIP PORT COVER**

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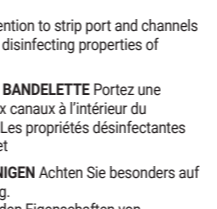
(EN) Items not manufactured by Universal Biosensors
CaviWipes™ (recommended), Alcohol Swabs, Sterile Gauze, Example of Lancets, Lint-free tissues/cloth

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Designed by
Universal Biosensors Pty Ltd
1 Corporate Avenue,
Rowville, 3178, Victoria, Australia

79017 v2.1.1
Date of issuance September 2022

MediMark Europe Sarl 11,
rue Emile Zola, BP 2332 F- 38033
Grenoble Cedex 2, France

UK Responsible Person:
Obelis UK Ltd
Sandford Gate, East Point Business Park,
Oxford OX4 6LB United Kingdom

©2022 Universal Biosensors. Xprecia Prime™ is a trademark of Universal Biosensors. All rights reserved. CaviWipes™ is a trademark of Metrex Research LLC. All other trademarks are the property of their respective owners.

XpreciaPrime™

Coagulation Analyzer

QUICK START GUIDE